


**Lab Name:**


1. Enter the Patient Identification information on the Collection Tube label.

**NOTE:** Do not allow fecal specimen to contact toilet water until after the sample has been collected. Do not urinate on fecal specimen or Collection Paper.


2. Attach Collection Paper to toilet seat.
  - a. Raise toilet cover.
  - b. Remove tape cover from each end of the Collection Paper.
  - c. Position Collection Paper on rear half of seat, allowing paper to sag.
  - d. Press the tape down on each side of the seat.
  - e. Deposit fecal specimen on Collection Paper.
  - f. Do not remove the Collection Paper from the toilet seat.

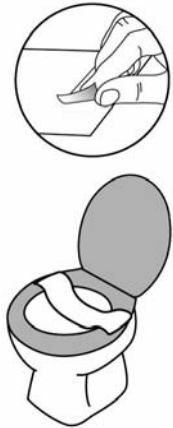


3. Unscrew the Sampler from the Collection Tube.

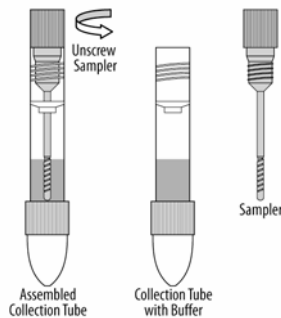

4. With the fecal specimen still on the Collection Paper, randomly pierce the specimen with the grooved end of the Sampler in **at least five (5) different sites**.

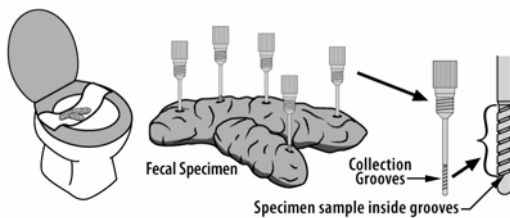

5. Insert the Sampler into the Collection Tube and firmly tighten it.

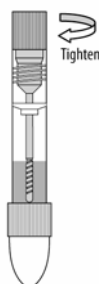

6. Shake the tube to mix the specimen and the FOB buffer.

