

cobas[®] PCR Urine Sample Kit

For *Chlamydia trachomatis* and *Neisseria gonorrhoeae*

Short guide¹ for urine sample collection

URINE SAMPLE FOR THE PATIENT

1. COLLECT:

Prior to sampling, the patient should not have urinated for **at least one hour** and **female patient must not clean** the labial area.



Ask the patient to provide the **first catch urine** (10-50mL) into a urine collection cup.



FOR HEALTHCARE PROFESSIONALS


2. PIPET:

Use the provided disposable pipet **to transfer** (within 24 hours) the **urine** into the **cobas[®] PCR Media tube**. **The correct volume of urine has been added when the fluid level is between the two black lines on the tube label.**

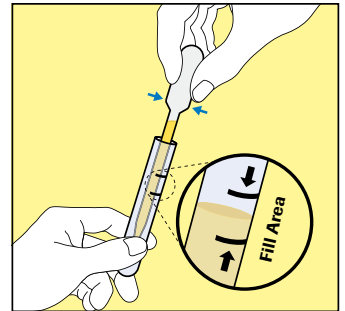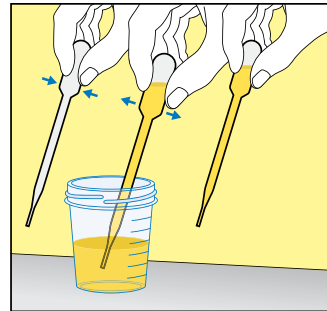

3. CLOSE:

Tightly re-cap the tube.

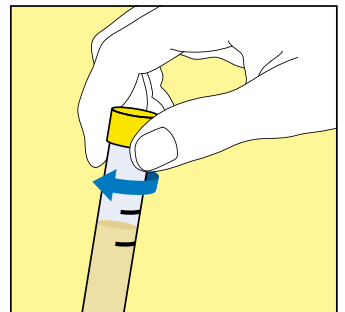


HANDLING PRECAUTIONS:
Use care to avoid splashing of contents.

4. MIX:

Invert the tube 5 times.

The specimen is now ready for transport.

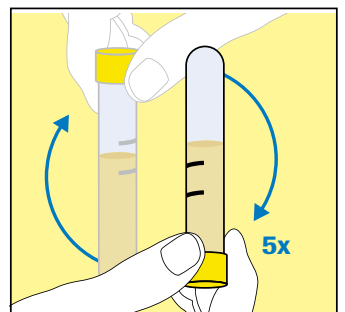

Transport: 2°C to 30°C ▪ **Stability of sample:** 12 months

THE SPECIMEN WILL BE REJECTED IF THERE IS:

- Inadequate volume (insufficient or excess)
- Excess of blood (>0.35%)
- Replens[®] vaginal moisturizer



¹ Individuals collecting or directing the collection of samples should consult the complete instructions for use.
COBAS is a trademark of Roche. © Roche 2017.