
Meatal Swab Specimen Collection Guide


Clinician-collection with the cobas® PCR Media Uni or Dual Swab Sample Kit

PROCEDURAL NOTES:

- If using the **cobas®** PCR Media Dual Swab Sample Kit, discard the flocked swab, as it is not needed for this procedure. **Use only the woven swab.**
- Avoid contact of the **cobas®** PCR Media with the skin, eyes or mucous membranes. If contact does occur, immediately wash with large amounts of water.

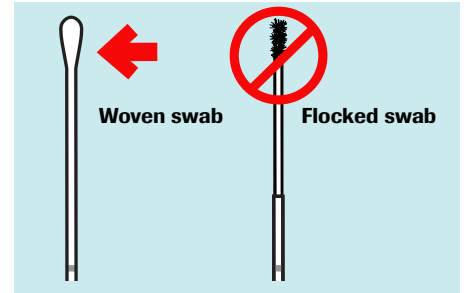
NOTE: Do not pre-wet the swab in **cobas®** PCR Media before collection.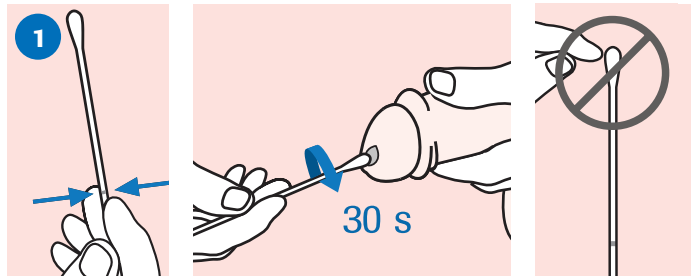


COLLECT:

- To collect the specimen, hold the swab with the scoreline above your hand.
- Turn the swab for about 30 seconds while rubbing the swab around the tip of the penis.
- Move the swab away carefully.
- Do not let the swab touch any other surface before placing it into the collection tube.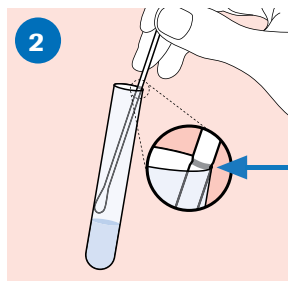


ALIGN:

- Remove the cap from the **cobas®** PCR Media Tube.
- Lower the swab specimen into the tube until the visible scoreline on the swab shaft is aligned with the tube rim.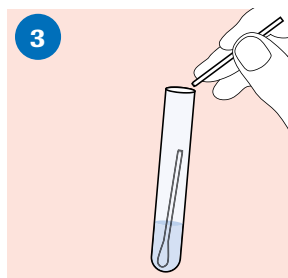

BREAK:

- Carefully leverage the swab against the tube rim to break the swab shaft at the scoreline.

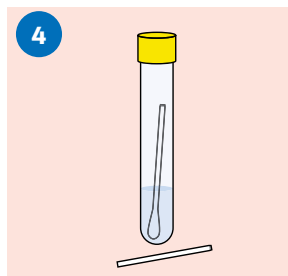CLOSE:

- Tightly close the **cobas®** PCR Media Tube. The specimen is now ready for transport.
- Discard the top portion of the swab.

SPECIMEN TRANSPORT AND STORAGE:

- Transport and store the **cobas®** PCR Media Tube containing the collection swab at 2°C to 30°C.
- The specimen should only contain 1 woven swab and may be rejected if the tube contains no swab or 2 swabs.

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